


Shipper: Single-use RBC Shipper (SKU # 1RBC10X72)

Blood Products: Blood Products (1-10°C)

Materials Needed: 48-hour Single-use shipper, (x4) EC4 gel packs and (x2) foam separators and (x1) payload insert tray

Packaging assembly illustration:

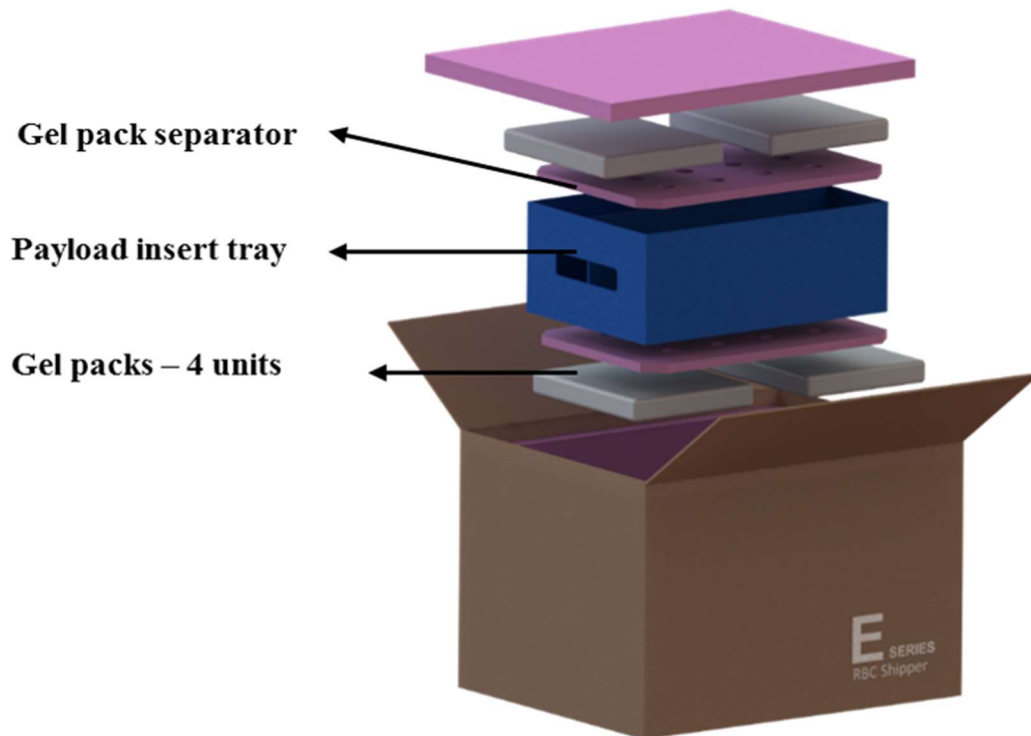

Coolant Pre-conditioning:

- Charge four (x4) EC4 Gel packs in a freezer (< -20°C) for a minimum of 24 hours.

Pack-out Instructions:

- 1) Place two frozen EC4 gel packs in the bottom of the container side-by-side.
- 2) Lay foam gel pack separator on top of the EC4 gel packs.
- 3) Place the payload insert tray into the container.
- 4) Load payload units into the payload insert tray and place the remaining gel pack foam separator on top.
- 5) Take the remaining two EC4 gel packs out of the freezer and place side-by-side on the top foam separator.
- 6) Close the container lid making sure that the lid is sealed properly and apply packaging tape