


**Shipper:** 48-hour One-way Refrigerated Shipper (SKU # 1RBC4X48)

**Blood Products:** Blood Products (1-10°C)

**Materials Needed:** MaxPlus 48-hour Shipper, (x2) RB402 bricks and (x2) RB60 bricks

### Packaging assembly illustration:

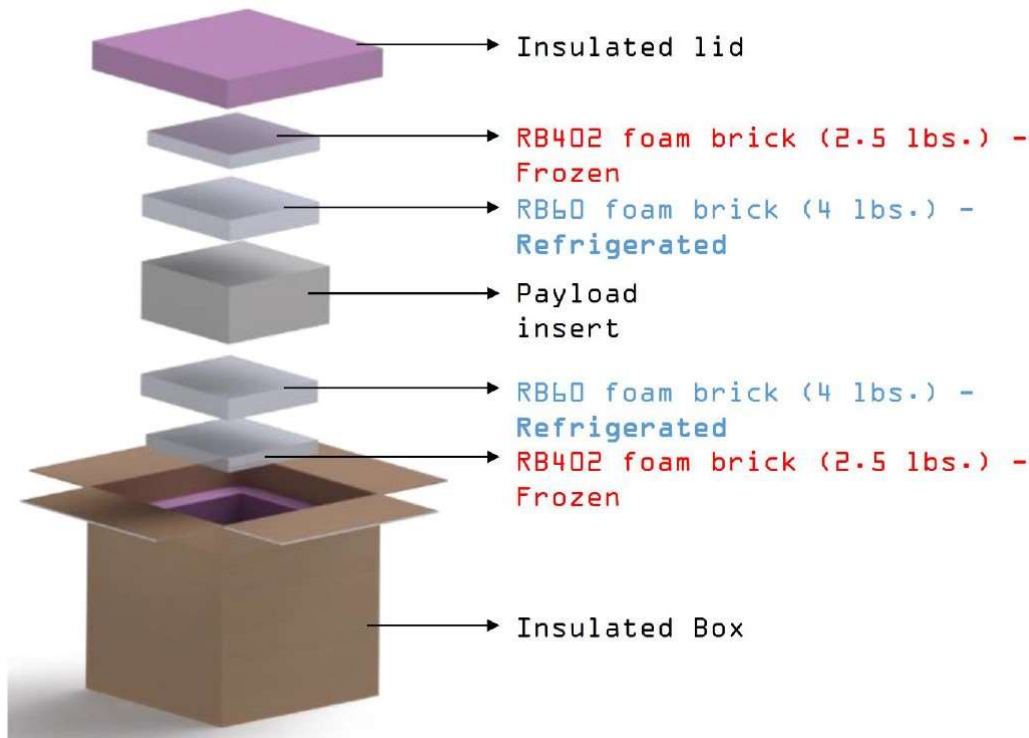

### Coolant Pre-conditioning:

- Charge (x2) RB402 bricks in a freezer at ( $< -20^{\circ}\text{C}$ ) for a minimum of 24hrs.
- Charge two (x2) RB60 bricks in a refrigerator at ( $1-6^{\circ}\text{C} \pm 1^{\circ}\text{C}$ ) for a minimum of 24 hours.

### Pack-out Instructions:

- 1) Place one frozen RB402 in the bottom of the container.
- 2) Place a refrigerated RB60 into the container.
- 3) Load blood products (max 4 of units) into the payload insert provided.
- 4) Close the payload insert and tape shut with packaging tape and set on top of the bottom RB60 brick.
- 5) Place a refrigerated RB402 brick on top of the payload insert.
- 6) Set a frozen RB402 brick directly on top of the refrigerated RB60 brick.
- 7) Lastly, seat the lid on top of the fully packed 48-hour system, close the flaps and secure with packaging tape. The shipper is now ready for transport.