


**Shipper:** MaxPlus Non-dry Ice Frozen Plasma Shipper (SKU#PL13V48-N).

**Blood Products:** Frozen Plasma (FFP, < -18°C)

**Materials Needed:** MaxPlus Non-dry Ice Frozen Plasma Shipper, (x6) PCM21N Plastic Cassettes

### Packaging assembly illustration:

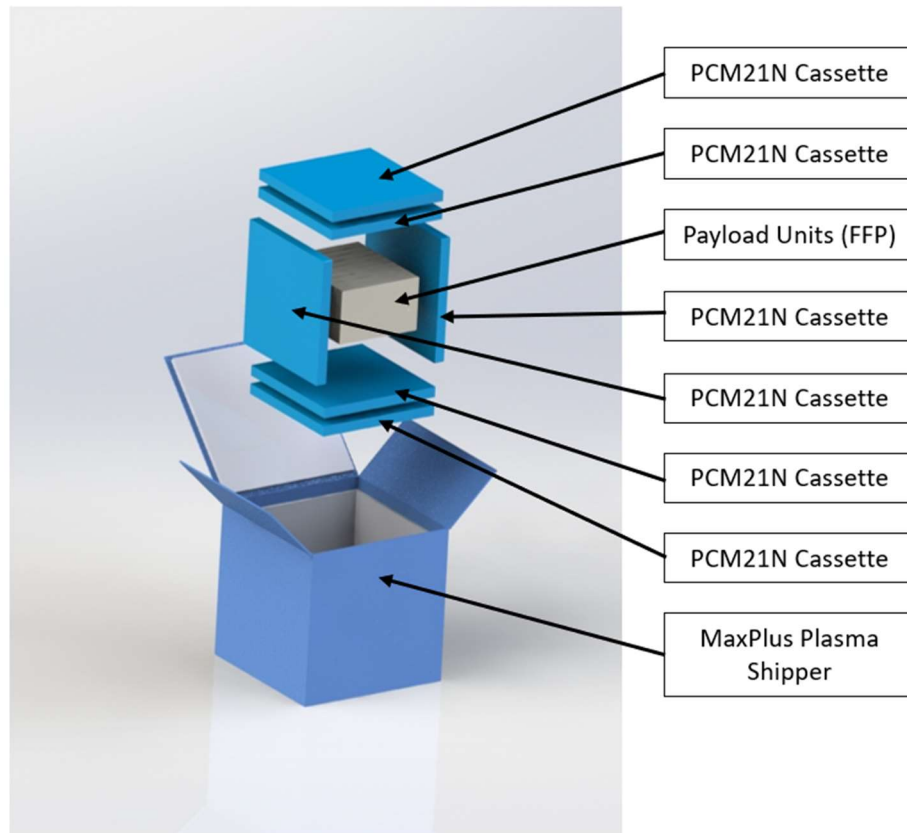

### Coolant Pre-conditioning:

- Charge six PCM21N phase change cassettes (blue plastic) in the freezer at -30°C or below for a minimum of 24 hours. Plasma freezers are acceptable.

### Pack-out Instructions:

**Step 1:** Place two frozen PCM21N phase change cassette (blue plastic) in bottom of the shipper.

**Step 2:** Place two frozen PCM21N phase change cassette (blue plastic) against the left and right walls of the shipper.

**Step 3:** Place the payload units directly on top of the bottom phase change cassettes.

**Step 4:** Place the last two frozen PCM21N phase change cassettes directly on top of the payload units. Fill any empty space inside the payload volume with packaging material (bubble wrap, etc.) to avoid payload shifting during shipment.

**Step 5:** Close and seal the lid properly using packaging tape.