


Shipper: Small sized MaxPlus Blood Cooler (SKU # RT10MB)

Blood Products: Platelets (20-24 °C)

Materials Needed

MaxPlus Blood Cooler, (x2) S10 Bottles, Payload Insert

Packaging assembly illustration:

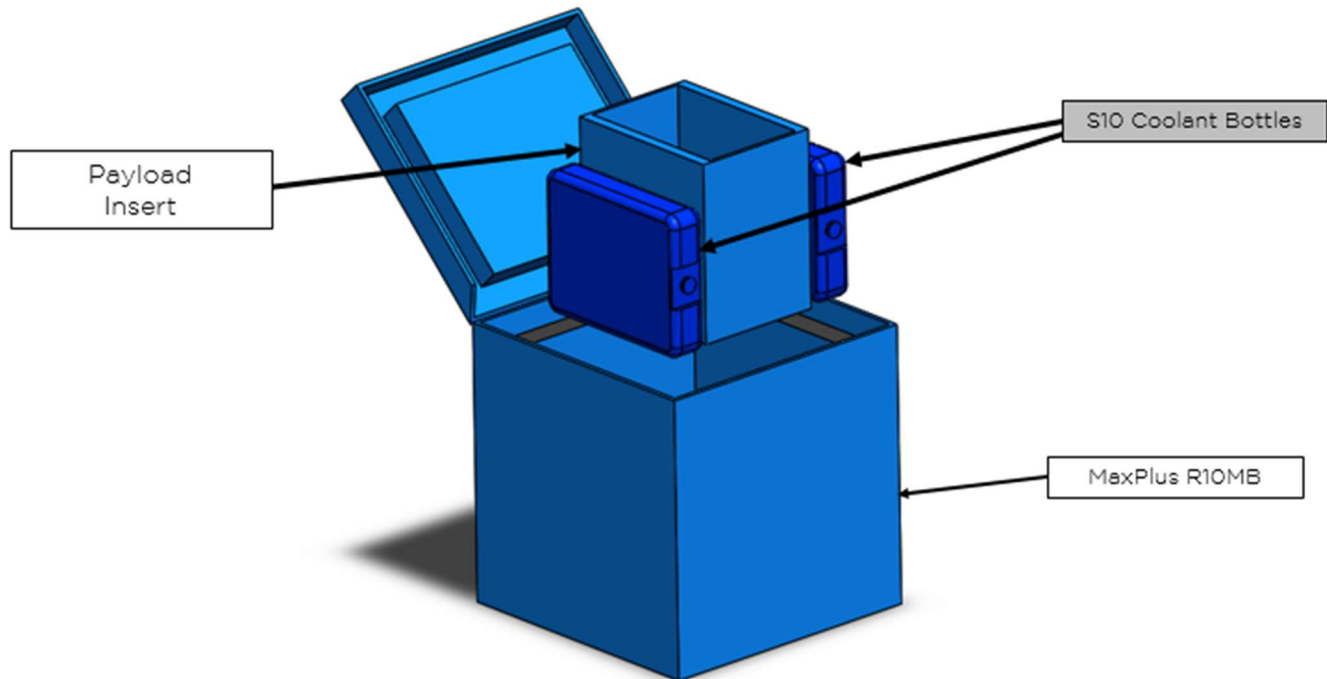

Coolant Pre-conditioning:

- Charge (x2) S10 coolant bottles in an incubator or controlled room temperature at (23°C ± 1°C) for a minimum of 24 hours.

Pack-out Instructions:

1. Place (x2) S10 coolant bottles against two opposite walls of the container (refer to package assembly illustration)
2. Place the payload insert into the cooler.
3. Place your payload units inside the insert and close the lid making sure that the lid is sealed properly.
4. The cooler is now ready to be issued.