


Shipper: MaxPlus Platelet Shipper (SKU # U17PLT30)

Blood Products: Platelets (PLTs, 20 – 24°C)

Materials Needed: MaxPlus Platelet Shipper, (x4) B18 PCM22 bottles

Packaging assembly illustration:

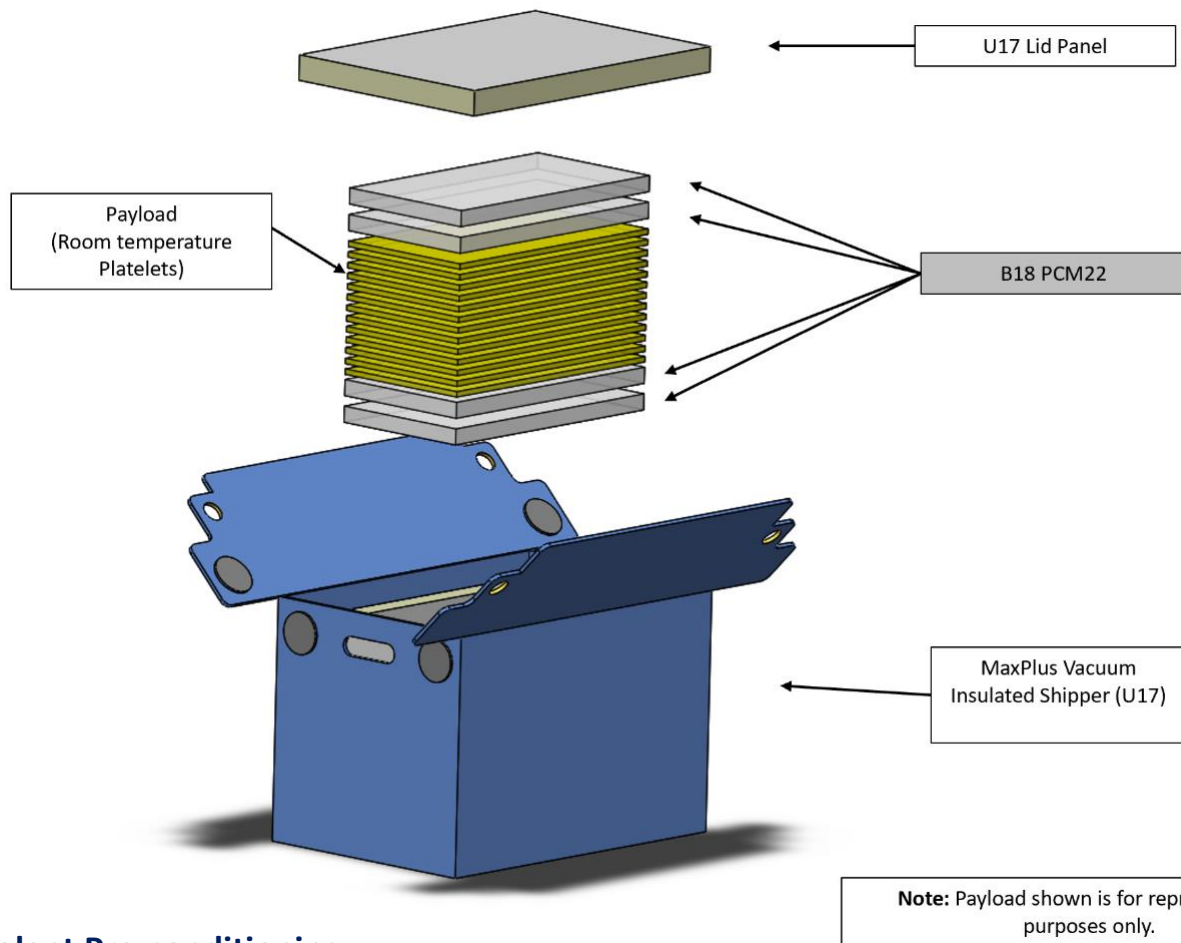

Coolant Pre-conditioning:

- Charge four B18 PCM22 coolant bottles in an incubator or controlled room temperature at $(23^{\circ}\text{C} \pm 1^{\circ}\text{C})$ for a minimum of 24 hours.

Pack-out Instructions:

- Place two B18 PCM22 coolants in the bottom of the container.
- Load desired number of platelet units directly on top of the bottom coolant bottles.
- Place both of the remaining B18 PCM22 coolants on top of the payload units
- Place the lid on top (**Note:** black gasket material facing down).
- Equip the tamper evidence solution to the left and right-side grommets and Velcro the flaps down.
- The container is now ready for transport. No packaging tape required.